ENGLISH

6

Warnings	6
Installation	7
Maintenance/Cleaning	9
Cleaning Chart	10
Reassembling	11
Warranty	11

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NOTE: Check all parts are free from shipping damage. In case of damage DO NOT use. Contact Sunrise Medical or your supplier for further instructions.

Sunrise Medical recommends that a clinician such as a doctor, therapist or rehab engineer experienced in seating and positioning be consulted to determine if a Jay Xtreme Active Cushion is appropriate. Cushions should only be installed by:

- an authorized Sunrise Medical supplier
- a qualified healthcare professional who has undergone appropriate training by either Sunrise Medical or their approved supplier.

⚠ GENERAL WARNING

- DO NOT install this equipment without first reading and understanding this manual. If you are unable to understand the Warnings, Cautions and Instructions, contact a qualified clinician or supplier - otherwise injury or damage may occur.
- Hook and loop Velcro® should not come into prolonged contact with moisture as this may degrade the adhesive and lead to a failure of the Velcro®.

INTRODUCTION

The Xtreme Active Cushion is designed with the benefits of skin protection in a light-weight, comfortable and low-maintenance design. The Xtreme Active incorporates a Jay Flow Fluid pad, a soft foam base, with an incontinent inner cover, optional lateral and medial thigh supports and an outer cover. For active and regular users, the Xtreme Active helps to provide pressure and shear reduction which may help reduce the incidence of skin breakdown. It is comfortable and facilitates movement, allowing the user to change positions on the cushion. The Jay Xtreme Active cushion is designed to help reduce the incidence of pressure ulcers. However, no cushion can completely eliminate sitting pressure or prevent pressure ulcers. The Jay Xtreme Active cushion is not a substitute for good skin care including proper diet, cleanliness and regular pressure relieving.

Maximum User Weight:

- 150 kg for all sizes.

⚠ WARNING

- Installing a cushion on a wheelchair may affect the centre of gravity of the wheelchair and may cause the wheelchair to tip backwards, potentially resulting in injury. Always assess the potential need for anti-tippers or amputee axle adapter brackets to be added to the wheelchair to help increase stability.
- Prior to prolonged sitting, any cushion should be tried for a few hours at a time while a clinician inspects your skin to ensure that red pressure spots are not developing. You should regularly check for skin redness. The clinical indicator for tissue breakdown is skin redness. If your skin develops redness, discontinue the use of the cushion immediately and see your doctor or therapist.
- The Xtreme Active cushion is designed to help reduce the incidence of pressure ulcers. However, no cushion can completely eliminate sitting pressure or prevent pressure ulcers.

⚠ WARNING

- Do not leave the cushion outside overnight at temperatures below 40° F (5° C). Allow the cushion to warm to room temperature before using. Sitting on extremely cold or hot surfaces can cause skin damage.
- Avoid sharp objects or exposure to excessive heat or open flame.
- **PROPER FLUID LEVEL:** The amount of fluid contained in the insert is determined by the clinician during a seating evaluation. This should not be adjusted without a clinician directive as it can alter the pressure relieving properties and lead to injury or damage.
- **OBSTRUCTIONS: DO NOT:** place any obstructions between the user and the cushion because it will reduce product effectiveness.
- **CUSHION AND COVER ORIENTATION:** Product must be used with the fluid insert facing up. If the cover is not used correctly or if the cover is the wrong size, it may reduce or eliminate the cushion's benefits and could increase risk to the skin and soft tissue.
- **CUSHION BASE AND FLUID PAD:** this product has two different fluid pads that match two different bases. The bases are not interchangeable. Using a fluid pad with the incorrect base will reduce or eliminate the cushion's benefits, increasing risk to the skin and soft tissue

CUSHION INSTALLATION

Select a cushion size which corresponds to your hip width when you are seated. The cushion length should extend 3-5 cm from the back of the knees. Place the cushion as far back as possible in the wheelchair. The curved rear edge and the cut-out corners should be as close as possible to the wheelchair back. In order to achieve the lowest possible seated pressure, the footrests should be set at a low position so that your thighs rest comfortably on the front part of the cushion. (Setting the footrests correctly can reduce pressure on certain points.) When you are seated on the Jay Xtreme Active cushion, your ischium should be centralised on the fluid pad. If the cushion is not positioned correctly, this can lead to bottoming out and tissue damage. If you have oblique posture, position the cushion slightly further forward in the chair, so that you are not sitting on the front foam part of the cushion. (If you have extreme oblique posture, we would recommend the JAY J2® and JAY J3 cushions.) In addition, if your backrest is a sling-type, then this should be replaced. Instead of this, you could use a JAY J3 backrest. A solid backrest positions the hips correctly and supports the trunk, so that you are sitting less on your sacrum, i.e. your pelvis is tilted backwards.

INSTALLATION OF DIFFERENT FLUID PADS

The Xtreme Active cushion can be ordered with two different fluid pads: one with a larger amount of fluid and another containing less fluid (optimized for lightweight).

The fluid pads are colour coded and need to be installed onto the inner cover according to the Velcro® loop and hook colours.

Fluid Pad L

Fluid Pad S

NOTE:

- The inner and outer covers of Xtreme Active do not have to be exchanged when you switch from one fluid type to the other. But the bases of the Xtreme Active are fluid-dependent.
- Always attach black hook on fluid pads to black loop on inner cover and grey hook on fluid pads to grey loop on inner covers.

CHECKING FOR BOTTOMING OUT

Bottoming out occurs on an Xtreme Active Cushion when you displace the fluid out from underneath your seat bones, which leaves you sitting on the foam base. It sometimes occurs on very thin individuals, people using recliner wheelchairs, people who slouch when sitting or if using a cushion that is too wide.

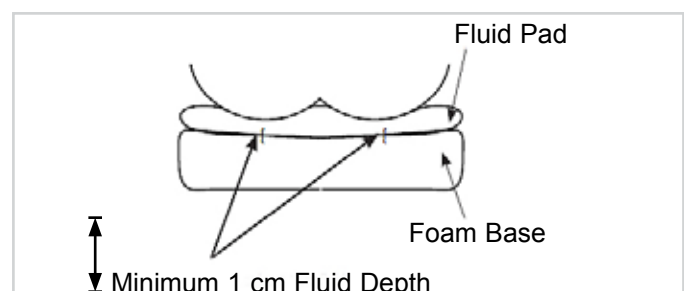
When bottoming out occurs, increased pressure is loaded onto the ischials and coccyx increasing the risk for skin breakdown.

To test for bottoming out, sit for at least 2 minutes on the Jay Xtreme Active cushion without the cover. Transfer up and off the cushion (or have someone help you transfer), trying not to disturb the fluid underneath you. Push down in the depressions on the pad where your ischials (seat bones) and coccyx (tailbone) were. You should have to push through at least 1/2" (1cm) of fluid before you feel the firm cushion base below.

If the cushion is properly positioned and the footrests are properly adjusted, and there is not at least the minimum 1/2" (1cm) of fluid, the cushion is bottoming out and should not be used. (If you have a relatively recent injury, a fluid layer of 2 cm is recommended, as this can possibly lead to post-traumatic atrophy). If bottoming out occurs, you might need an overfilled fluid pad. You can obtain an overfilled fluid pad from your dealer or from Sunrise Medical customer service.

⚠ WARNING

When bottoming-out occurs, increased pressure is placed onto the ischium and coccyx increasing the risk of skin breakdown. Immediately discontinue use of the pad. See your healthcare professional.

USE WITH A SOLID SEAT AND A SOLID BACKREST

Avoid using the cushion on overly stretched seat upholstery. Stretched upholstery does not provide adequate support for the cushion and improper positioning may result. Replace stretched upholstery or purchase the Jay Adjustable Solid Seat or the Jay Solid Seat insert that is appropriate for the cushion.

Avoid using overly stretched back rest upholstery. Stretched back upholstery does not provide adequate support for the body and improper positioning may result. Replace stretched back rest upholstery or purchase a back support product from Sunrise that is appropriate for you. Contact an authorized supplier for a free demonstration of these products.

⚠ WARNING

- When you install a seat cushion in a wheelchair, this affects the centre of gravity and could increase the risk of the wheelchair tipping over backwards!
- Always have the anti-tip tubes fitted to the wheelchair!

For better posture and to give greater relief from back pain, we recommend the JAY J3 backrest. Your Sunrise Medical dealer will be pleased to demonstrate these backrests for you.

SETTING THE LEG POSITION

In order to achieve the lowest seated pressure and maximise the leg support, lower the footrests, raise the seat and/or use a longer cushion. The leg wedges in the cushion base cover stop the legs from gradually moving apart. You can make the wheelchair sling tighter or consider using a solid seat insert. If the legs are still too far apart, consider a JAY J2, Jay Balance or JAY J3 cushion.

XTREME ACTIVE CUSHION ACCESSORY PARTS

Each Xtreme Active can accommodate leg support wedges (2 lateral thigh supports, 1 medial thigh support). These supports can be ordered as a kit and fitted through Velcro®s on the inner cover.

When positioned on the inner cover, the supports enhance the top surface contour of the cushion providing lateral thigh support without compromising softness. This increases the contact with the legs to help control abduction (separation of legs) and adduction (if medial thigh support is used).

Fluid Pad L and Leg Support Kit

Fluid Pad S and Leg Support Kit

EASY MAINTENANCE & CLEANING

- Monthly cleaning and regular maintenance may help extend the life of your cushion. During cleaning, component inspection is recommended.
- Check the cover for tears and excessive wear and replace if ripped, torn or otherwise not fully functional. Inspect the fluid pad for punctures or any other abnormalities.
- Check the foam insert to ensure foam consistency. While checking the fluid pad if you ever feel the fluid is firmer in one area, simply knead the fluid back to its original consistency and/or call your supplier.

TO CLEAN COVER

- Remove the cover from the foam insert and turn inside out.
- Machine wash in warm water (70°) and air dry.
- No ironing required.

⚠ CAUTION

- Do not dry clean covers or use industrial washers and dryers to clean cover.
- Do not steam autoclave.
- Do not bleach.
- To refit cover after washing, reinstall the cover on the foam insert. Ensure the back of the cover matches up to the back of the foam insert.
- If used in an institutional setting, write the resident's name with a permanent marker on the content label of the covers. This will help to ensure the return of the covers after washing.
- Use only Jay covers on Jay cushions. The cover is an important part of allowing the immersion into the fluid insert which redistributes the pressure. Changing the cover will alter this design and its efficacy.

CLEANING THE FOAM BASE / LEG SUPPORT WEDGES

Remove the covers and fluid pad from the foam base. Wipe lightly with a damp cloth. Do not use soap. Do not submerge in water. Wipe off with a clean cloth. Let it completely air-dry before reassembly. Remove the covers from the leg supports. The covers can be machine washed (70°C) and air dried. The leg support wedges can be wiped with a damp cloth (do not use soap and do not submerge in water) and then wiped off with a clean cloth. Let it completely air-dry before reintroducing them into the covers.

⚠ CAUTION

Avoid submersion of the foam base and leg supports in water.

CLEANING THE FLUID PAD

Remove the outer cover from the cushion. Remove fluid pad from the inner cover and wipe it with warm water and soap. Rinse with a clean, damp cloth, then wipe dry with a clean cloth.

⚠ CAUTION

- Never submerge the fluid pad in any liquid.
- Avoid harsh cleaning or rough handling as this may lead to a degradation of the fluid insert and lead to a leakage.

Should there be evidence of wear and tear on the cover, foam insert, fluid insert or other parts or if you notice that the consistency has deteriorated, please contact your dealer or Sunrise Medical for an assessment and further instructions.

LAUNDRY CARE SYMBOLS	
	Machine wash in warm water (70° C)
	Only non-chlorine bleach, when needed
	Drip dry
	Do not iron
	Do not dry clean

Approved Cleaning Procedures for Jay Cushions									
Cover	Jay Lite	Jay J3	J2	Jay Easy Visco	Jay Soft Combi	Jay Xtreme Active	Jay Basic	Jay Easy	Jay Balance
Washing chemo-thermal procedure 70°C 									
Cushion core (foam base)									
Steam autoclave up to 105 °C (1) 									
Spray and wipe disinfection 									
Immersion disinfection 									
Cushion elements									
Roho Air element	NA			NA	NA	NA	NA	NA	
Positioning elements	NA			NA	NA		NA	NA	
Fluid pad	NA			NA	NA		NA		
(1) An autoclave is a self locking machine that sterilizes with steam under pressure.									
The product warranty shall not apply to products damaged due to the failure to follow these cleaning instructions or the use of inappropriate cleaning compounds.									

Recommended Disinfectants

All disinfectants which are listed and recommended by the Robert Koch Institute:
http://www.rki.de/EN/Home/homepage_node.html

REASSEMBLING THE XTREME ACTIVE CUSHION AFTER CLEANING

Follow cleaning and maintenance instructions as described in “Easy maintenance, cleaning and re-use” then reassemble as follows.

Insert the foam base into the water resistance inner cover. Ensure that the zipper is at the rear of the foam base where the seat well is located, and the Velcro® is on top in the seat well. Attach the fluid pad to the inner cover at the seat well matching all Velcro® strip locations, according to the colour codes.

If your cushion contains leg thigh supports, place them on the inner cover according to your positioning needs. The Xtreme Active components are labelled with front and rear to help match components for easy assembly.

WARNING

- Insert the Jay Fluid pad on the inner cover making sure that all Velcro® strip locations are aligned according to the Velcro® loop and hook colours . A misalignment could cause an unwanted pressure point which could lead to injury.

WHEN TO CHECK

Check your cushion and fluid pad every month when you clean it.

JAY XTREME ACTIVE CUSHION WARRANTY

Each Jay fluid cushion is carefully inspected and tested to provide peak performance. Every Jay fluid cushion is guaranteed to be free from defects in materials and workmanship for a period of 24 months from the date of purchase, provided normal use. Should a defect in materials or workmanship occur within 24 months from the original date of purchase, Sunrise Medical will, at its option, repair or replace it without charge. This warranty does not apply to punctures, tears or burns, nor to the cushion's removable cover.

The removable cover is guaranteed to be free from defects in materials and workmanship for a period of six (6) months from the date of purchase, provided normal use. Should a defect in materials or workmanship occur within six (6) months from the original date of purchase, Sunrise Medical will, at its option, repair or replace it without charge.

Claims and repairs should be processed through the nearest authorized supplier. Except for express warranties made herein, all other warranties, including implied warranties of merchantability and warranties of fitness for a particular purpose are excluded.

ADDENDUM TO WARRANTY USE AND CARE GUIDE

Consult your health care professional for proper fitting and use, and read the following statements carefully:

- The fluid in this cushion may lose volume over time.
- Volume loss in this cushion may result in bottoming-out of the cushion.
- Monthly, during routine cleaning and inspection, check the cushion for bottoming-out (see instructions under “checking for bottoming-out” in the warranty use and care guide) and any fluid inconsistencies. If the fluid is firmer in one area simply knead it back to its original consistency.
- If bottoming-out occurs discontinue use of this cushion and contact your health care professional. If further assistance is necessary, contact Sunrise Medical Customer Services.

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©2011 Sunrise Medical (US) LLC
Sunrise Medical Pty. Ltd.
Sunrise Medical GmbH & Co. KG